



www.preat.com
800-232-7732

244 Blue Polishing Paste

Technical Information



244 Blue Polishing Paste

Preat 244 Blue Polishing Paste is the finest single step polishing compound available. Water soluble for easy cleaning and elimination of staining to produce a high shine on acrylic, gold, composites, non-precious, and chrome cobalt alloys.

244 Blue is a high shine polish in stick form for polishing all acrylic denture base materials, flexible denture base materials, chrome cobalt alloys, gold alloys and composites. The polish may be used on felt wheels and cones, rag wheels, and buffs.

Best results are achieved when used with either a goat hair and micro fiber mounted brush in a handpiece or lathe wheel.

The polish should be used at a slow speed of 1000 to 5000 rpm at a low to moderate pressure to eliminate excessive heat.

It is recommended that the user wear breathing and eye protection for personal safety.

T0095.REV00