



www.preat.com
800-232-7732

Brass Pencil



Technical Information

The holder for the brass pencil works like a mechanical pencil, and by turning the end of the pencil the user can extend or reduce the exposed brass insert. A new brass refill insert may be inserted into the pencil when needed by turning as well.

The brass insert is used to remove debris and clean burs and other cutting instruments. The brass is soft and may leave a small brass residue.

The user should use this instrument **in front of a vacuum and wear both protective glasses and breathing masks** to eliminate either breathing or absorbing the airborne brass particles.

T0098.REV00