




PERMA BLOCK

Preat Corporation
800-232-7732/805-693-8666
FAX: 805-693-8106
www.preat.com

Single Dose Cartridge

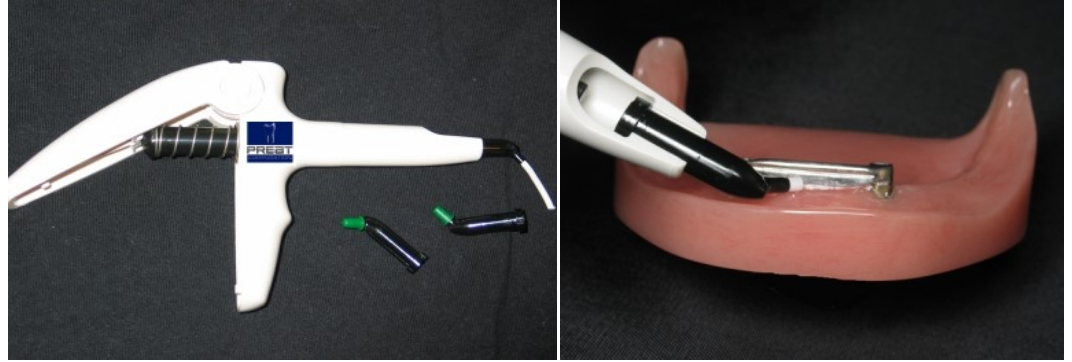

Perma Block is a specially designed, cellulose-based block-out material that is delivered in an easy to use single dose cartridge! Perma Block will adhere to a wet rubber dam, gingival and mucosal tissues and teeth. It will adhere under water and saliva!

Convenient. Saves Chairside Time. No mixing, no curing, no setting, no working times required.
Material is immediately placed, used and is easy to remove.

Perma Block is used to block out undercuts associated with large gingival embrasures, implant bars, precision attachments, or fixed partial denture pontics during impression making procedures. It's heavy body prevents displacement during intraoral pick-up of attachments with autopolymerizing resins. May be used for Laboratory duplication, but not indicated for Laboratory / Indirect Processing of Acrylic.

Procedure

The single dose cartridge fits into your Centrix or composite gun and may be rotated to any angle for ease of use. This provides a mechanical advantage for easy expressing and placement of Perma Block.

Perma Block is easily contoured or shaped with a wet instrument, cotton swab, or a wet finger in a rubber glove. Perma Block is easy to remove and clean up. Simply remove large sections or pieces with an instrument. Water and a brush will readily remove any remaining Perma Block material.



Package of 10 Single Dose Cartridges
Centrix Gun for Single Use Cartridges

\$36.00
\$60.00